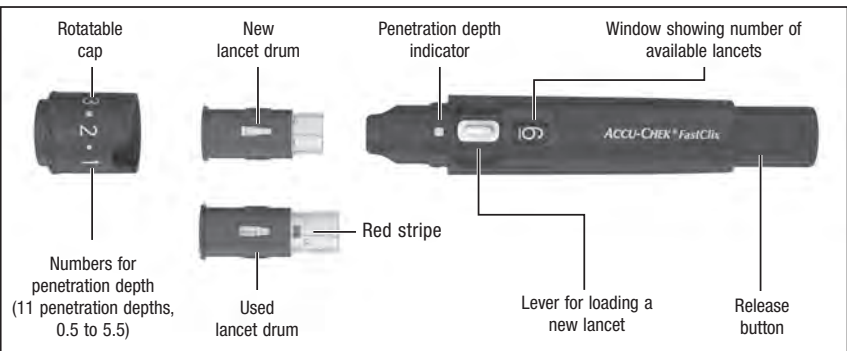
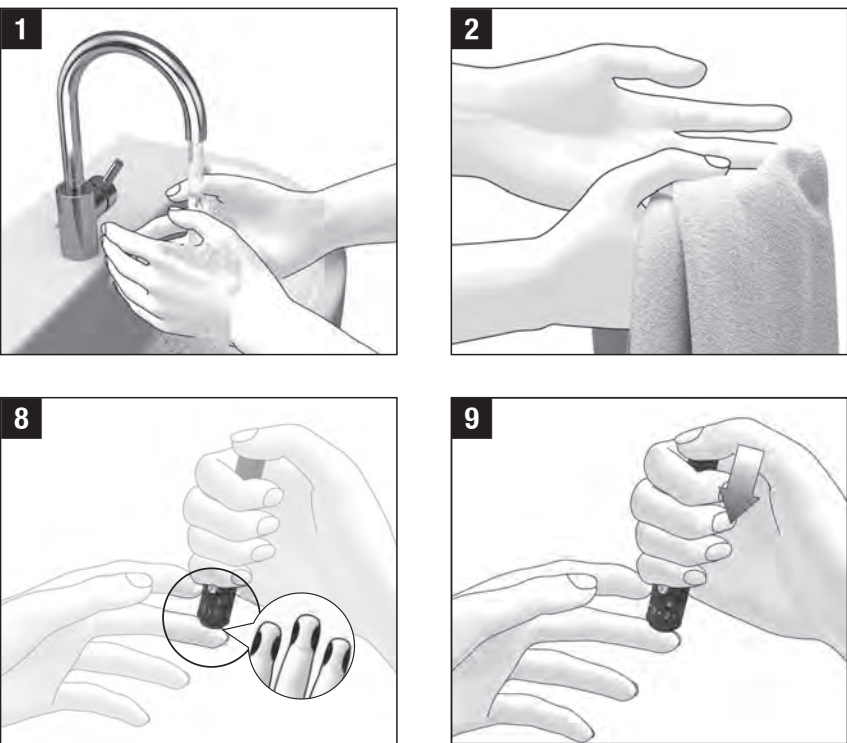
Instructions for Use

Read these instructions for use before you obtain a blood drop from your fingertip.

Intended Use

Finger Pricker

The Accu-Chek FastClix finger pricker is a device for simple, virtually pain-free collection of capillary blood from the fingertips. The Accu-Chek FastClix finger pricker is intended for single patient use only! It is only allowed to be used to obtain blood from one person. There is a risk of infections being transmitted if the finger pricker is used by other people, even by family members, or if healthcare professionals use this finger pricker to obtain blood from different people. This finger pricker is therefore not suitable for professional use in healthcare facilities and institutions. It is not a sterile product.

People in the area of diabetes self-monitoring are considered as target group.

Lancet

Six Accu-Chek FastClix lancets are in a drum. The lancet drum is intended to be used with the Accu-Chek FastClix finger pricker. The lancet drum is a disposable part. Each lancet in it is intended to be used only once. The Accu-Chek FastClix lancet drum includes 6 lancets and is intended to be used with the Accu-Chek FastClix finger pricker for simple, virtually pain-free collection of capillary blood. The lancet drum is a disposable article with 6 lancets each intended to be used only once. Each lancet has a protective cap that guarantees sterile condition until its first use. The lancets are sterilized using irradiation. The used lancet drum is protected against re-use.

General Safety Information

The instructions for use contain warnings and precautions:

A **WARNING** indicates a foreseeable serious hazard.

A **PRECAUTION** describes a measure you should take to use the product safely and effectively or to prevent damage to the product.

WARNING

Risk of suffocation

This product contains small parts that can be swallowed.

Keep the small parts away from small children and people who might swallow small parts.

Risk of injury

If the penetration depth is too deep, the puncture can injure children. Before using the finger pricker for the first time on children, ask your healthcare professional. Start with the lowest penetration depth.

Risk of infection

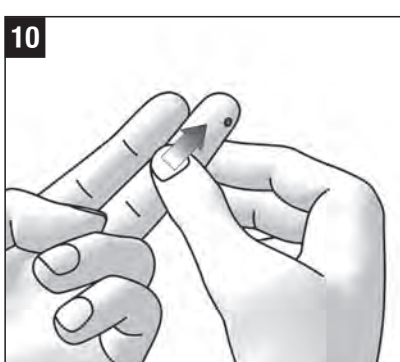
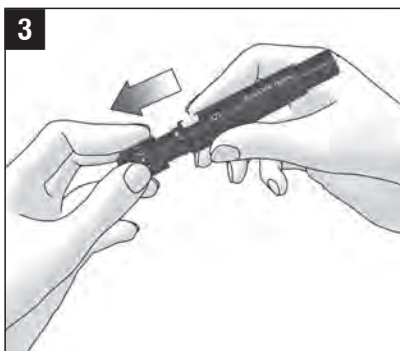
If the finger pricker or the lancet drum has come into contact with blood, it can transmit infections. The finger pricker and the lancet drum must not be used by other people, even by family members. Healthcare professionals must not use the finger pricker to obtain blood from different people.

If the finger pricker or lancet drum was dropped or exposed to other mechanical stress, check it for damages, for example, cracks. If the finger pricker or the lancet drum is damaged or if a lancet protrudes from the lancet drum, do not use the finger pricker or the lancet drum.

PRECAUTION

Risk of injury

- Skin contact to the finger pricker, lancet drum, or lancet can cause skin irritation.

If your skin becomes irritated or inflamed, consult your healthcare professional.

- If the lancet is pricked into a hard surface and not into the skin, the needle can be damaged. Only use the lancet to obtain a blood drop.
- Parts of the finger pricker, for example, the release button, can shoot away during use. Do not point the finger pricker at your face or other people.

If you find lancets in the packaging, do not use them. Discard them. You can use all lancet drums from the packaging. If you happen to insert a lancet drum with missing lancets, you cannot obtain blood each time you load a new lancet. In this case, simply load the next lancet.

Only insert Accu-Chek FastClix lancet drums into the Accu-Chek FastClix finger pricker. Other lancets or lancet drums may impair the operation of the finger pricker or damage it permanently.

Tips and Tricks

You must insert the lancet drum completely into the finger pricker. Only then, you can place the cap correctly on the finger pricker. The lancet drum is completely inserted when only the projecting edge of the lancet drum protrudes from the front of the finger pricker. The cap is placed correctly on the finger pricker when you push it as far as it goes. You must hear and feel the cap click into place. Do not use any force to place the cap on. Force can damage the cap and the finger pricker. Correct the position of the lancet drum instead.

Do not insert a lancet drum and press the release button at the same time. Do not insert a lancet drum with the release button resting on a surface, for example, a table top. This could release a lancet which could inadvertently prick you.

Do not remove the lancet drum from the finger pricker before all 6 lancets are used. You cannot re-insert the lancet drum.

When you press the release button, the finger pricker releases a lancet. You do not need to prime the finger pricker beforehand.

If you have no experience with this finger pricker, start with a low penetration depth, such as penetration depth 2.

In general, blood can be obtained from any finger. Certain fingers can be unsuitable, if for example, a skin or fingernail infection is present. Obtain capillary blood from the sides of the fingertips as these areas are the least sensitive to pain.

If not enough blood emerges, apply more pressure on the finger pricker when you next obtain blood. If that is not sufficient, increase the penetration

depth progressively in addition. If too much blood emerges, decrease the penetration depth.

Using the Lancet Drum

WARNING

Risk of infection

- If a lancet was already used, it can transmit infections. Use a lancet only once to obtain blood.
- If the lancet drum is damaged, the lancets can be unsterile and lead to infection.

Visually inspect the lancet drum. If the lancet drum is deformed, cracked, or otherwise damaged, do not use it. Discard the lancet drum.

- In rare cases, the lancet is not pulled back into the finger pricker after you obtained blood. Check that the lancet does not protrude after you obtained blood. If the lancet protrudes, carefully set the finger pricker aside. Keep the finger pricker out of reach of other people and contact customer support.

PRECAUTION

Risk of infection

- A puncture site that was not cleaned can lead to infection.

Before obtaining blood, clean the puncture site.

- If the use by date of the lancet drum has expired, the lancets can be unsterile. Use only lancet drums that are within the use by date. The use by date is printed next to the  symbol on the packaging.

Follow the instructions that are shown in the illustrations.

Test your blood glucose immediately after you obtained blood, according to the instructions for use for your blood glucose meter.

Operating Conditions

Use the finger pricker including accessories under the following operating conditions:

- Temperature range: +5 to +45 °C
- Relative humidity range: 5 to 90 %

Storing the Finger Pricker and the Lancet Drums

Do not store the finger pricker and lancet drums at extreme temperatures, for example, in a hot car. This can impair the operation of the finger pricker and lancet drums.

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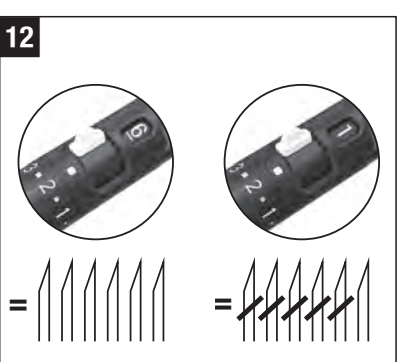
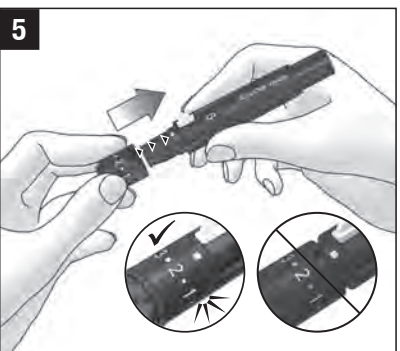
Cleaning and Disinfecting the Finger Pricker and the Cap

PRECAUTION

Risk of infection

If the finger pricker or the cap has come into contact with blood, it can transmit infections. Clean and disinfect the finger pricker and the cap always before someone else handles them, for example, to assist you.

To prevent infections from being transmitted, you must clean and disinfect the finger pricker and cap regularly:

- At least once a week
- When there is blood on the finger pricker or cap
- Always before someone else handles the finger pricker, for example, to assist you.

You need 3 lint-free cloths, water and 70 % isopropanol. The cloth used for disinfection must be large enough to wrap the finger pricker in. Before you disinfect the finger pricker and the cap, you first have to clean them to remove blood and other dirt.

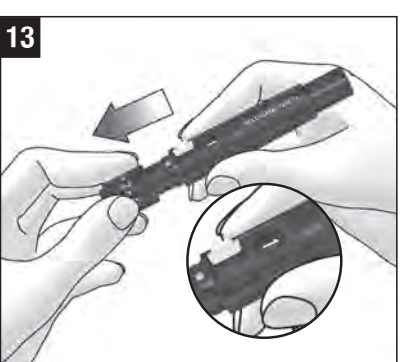
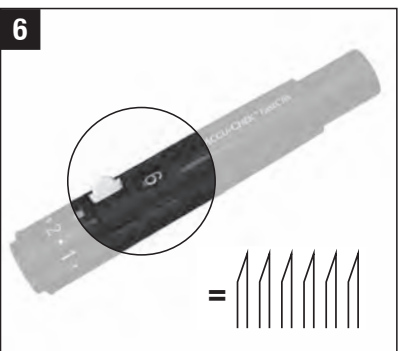
To avoid damaging the finger pricker and the cap:

- Do not dip the finger pricker and the cap in the fluids.
- Use only water for cleaning and only 70 % isopropanol for disinfecting.

- Remove the cap from the finger pricker.
- Remove the lancet drum from the finger pricker.
- For cleaning, moisten the first cloth with water.
- Wipe the finger pricker and the cap thoroughly with the moist cloth. Wipe the finger pricker from the outside only and the cap from the outside and the inside.
- Dry the finger pricker and the cap with a second, dry cloth.
- For disinfection, dampen a third cloth with 70 % isopropanol.
- Wipe the finger pricker and the cap thoroughly with the isopropanol cloth for 2 minutes each. Wipe the finger pricker from the outside only and the cap from the outside and the inside.
- Place the cap back on again.
- Wrap the whole finger pricker in the isopropanol cloth.
- Leave the finger pricker wrapped for 8 minutes.
- Unwrap the finger pricker and let it dry in the air.

Discarding the Used Lancet Drum

If no other regulations apply in your country, used lancet drums can be discarded in household waste.

Discarding the Finger Pricker

PRECAUTION

Risk of infection

If the finger pricker has come into contact with blood, it can transmit infections.

Discard the used finger pricker as infectious material according to the regulations applicable in your country.

Discard an unused finger pricker as plastic waste. For information on how to discard the finger pricker correctly, contact your local council or authority.

Alternative Sites for Obtaining Blood

You can obtain capillary blood for blood glucose testing not only from the fingertips, but also from other sites (palm, forearm, upper arm, calf, or thigh), so-called alternative site testing (abbreviated as AST). To be able to obtain capillary blood from other sites, you have to replace the regular cap by a special cap, the Accu-Chek FastClix AST cap.

Ordering Information

If you want to order an Accu-Chek FastClix finger pricker, Accu-Chek FastClix lancet drums or an Accu-Chek FastClix AST cap, contact the Accu-Chek Customer Careline (see section *Customer Support and Service Centre*).

Reporting of Serious Incidents

For a patient/user/third party in the European Union and in countries with identical regulatory regime; if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Last Update

2022-05

Customer Support and Service Centre

United Kingdom

Roche Diabetes Care Limited
Charles Avenue, Burgess Hill
West Sussex, RH15 9RY, United Kingdom
Accu-Chek Customer Careline¹⁾
UK Freephone number: 0800 701 000
ROI Freephone number: 1 800 709 600
¹⁾calls may be recorded for training purposes
Some mobile operators may charge for calls to these numbers.
www.accu-check.co.uk
www.accu-check.ie

EN DA NO SV FI

ACCU-CHEK
FastClix

Finger pricker and lancet

Fingerprikker og lancet

Blodprøvetaker og lansett

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
DE 09721452001(01) – US 09721452001A

	Consult instructions for use or consult electronic instructions for use
	Caution, refer to safety-related notes in the instructions for use accompanying this product.
	Use by
	Sterilized using irradiation
	Use only once
	Single patient multiple use
	Wash your hands
	Date of manufacture
	Medical device
	Manufacturer
	Unique device identifier
	Catalogue number
	Batch code
	Complies with the provisions of the applicable EU Legislation

CE 0123

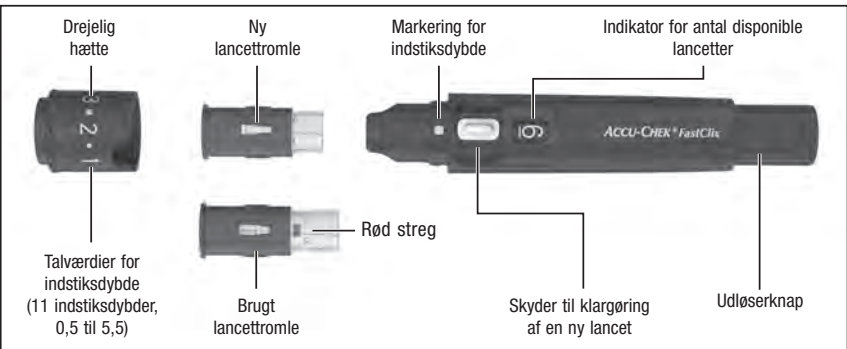ACCU-CHEK and FASTCLIX are trademarks of Roche.
© 2022 Roche Diabetes Care
Roche Diabetes Care GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany
www.accu-check.com

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